



If you are currently using a MiniMed 780G pump: Identify your current software version

1. From Home screen, press the up arrow to get to the Status screen
2. Select Pump
3. Press the down arrow to get to Software (it's at the very bottom)

If updating from version 6.7, read sections 1 and 2.
6.23, read sections 2.


Note: All these changes will be available after completing the software update to version 6.42

Section 1



Bolus feature allows manual BG[†] entry

Less steps to enter a BG reading into the pump. The BG reading entry can now be entered in the Bolus Wizard™ and SmartGuard™ bolus feature.

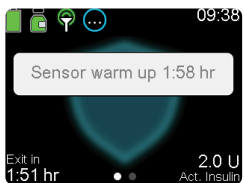

Sensor warm-up banner

A countdown timer displays on your Home screen during Sensor warm-up.




Reservoir and Set Change

Change just the Reservoir or just the Set with these new options.



Set Change reminder


Standard infusion set and reservoir changes are done every 2-3 days Medtronic Extended™ infusion set and reservoir can be worn up to 7 days. The wear duration option includes 2-7 days.

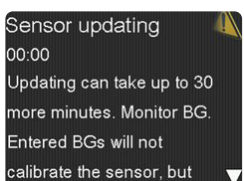

Sensor updating message


"Sensor updating" is the system confirming the reliability of the sensor; most sensors will resume normal function with no action needed. The message now provides the amount of time the sensor updating will take.

Section 2

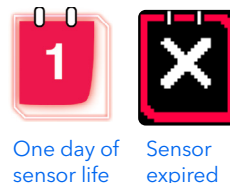


Sensor life icons

Appear on the Home screen. At any time, check the Status screen to see how many days of sensor life remain.

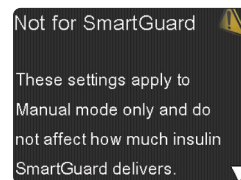


Not for SmartGuard™

A reminder message that the adjustments to Basal Pattern Setup and Max Basal setting only impact insulin delivery when in Manual mode. These settings have no impact on SmartGuard™ feature.

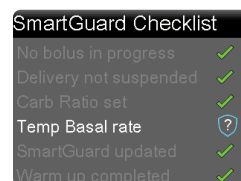

SmartGuard Checklist

The updated checklist provides more details on what is needed to enter the SmartGuard™ feature.

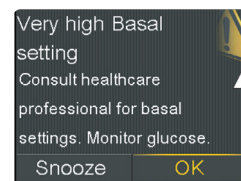

Very high Basal setting

A message will appear when the system calculates that the Manual mode Basal Pattern will deliver more insulin than typically needed by the user.

† BG = blood glucose
‡ Fingersticks required in manual mode & to enter SmartGuard.
If symptoms don't match alerts and readings, use a fingerstick. Refer to user guide.

Disclaimer
Information contained herein is not medical advice and should not be used as an alternative to speaking with your doctor. Discuss indications, contraindications, warnings, precautions, potential adverse events and any further information with your health care professional. AU-DBA-2600033 © 2025 Medtronic. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc.™™ Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company