


Getting started with the CareLink™ Connect app

Compatible phone? Check your device at <https://www.medtronic-diabetes.com.au/mm780g-supported-devices>.




1. Open the app and follow the on-screen instructions. You may be warned that your device is untested but you can still continue.

2. Log into your CareLink™ Personal care partner account. Sign up if you do not already have one

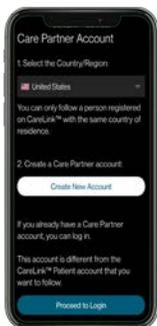


Note: Your country must be the same as the person you will be following


3. Read the Warning screen and accept by tapping Next

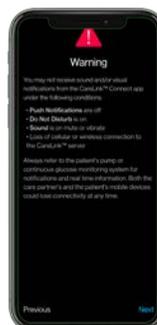

4. Enter the CareLink™ Personal username of the person you wish to follow*

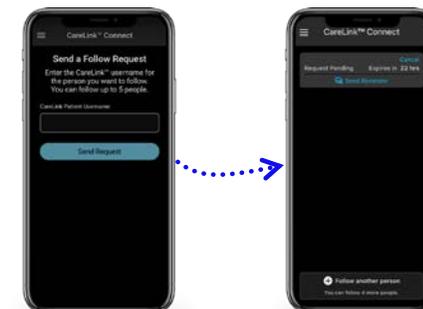

5. Once accepted, you will be prompted to set up your notifications. You will not receive any notifications** until these are set

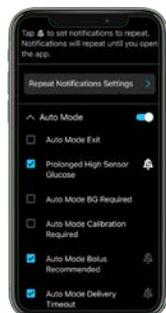

6. Tap **Follow another person** to add up to 5 MiniMed™ Mobile app users to your account



Each link will require notifications setting up

They must accept your connection request in their CareLink™ Personal account within 24hrs

Tip: For an improved experience, automatic updates to mobile operating systems should be deactivated - if one becomes available, check your local website to see if it is supported before installing

Representative patient profile, Google Play is trademark of Google LLC and App Store is a trademark of Apple Inc.

*See local website for supported devices - must be using and logged into the MiniMed Mobile app

**Internet connection required for live data and notifications - Phone screens may look different depending on the pump system and mobile phone used.

Linking to multiple care partners



CareLink™ Connect app users can follow up to 5 care partners using the MiniMed™ mobile app. You can switch between live views and receive push notifications from them all, as set by you.



Notifications



Notifications are colour coordinated:

- Alarms are red
- Alerts are yellow
- Reminders are grey
- Messages are blue

Each care partner that you are following must have notifications set up individually

Important Information:


- Notifications must be set in the Settings menu before any will be received.
- Having your phone on **Do Not Disturb**, **Mute** or **Vibrate** may cause you to miss notifications

Notification privacy




You can select if you want to see health related information in your notifications or have it hidden.

Privacy option switched on:

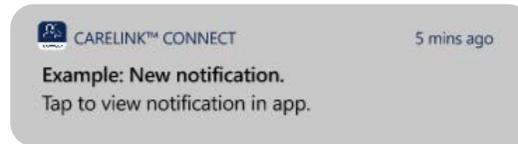


Or

Privacy option switched off:

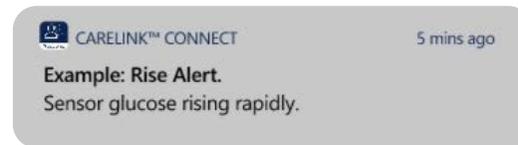

Repeat notifications:

- Some notifications can be set to repeat when received at certain times you specify if they have this symbol
- The notification sound can be personalised and will sound for 30 seconds and repeat every 60 seconds until the app is viewed

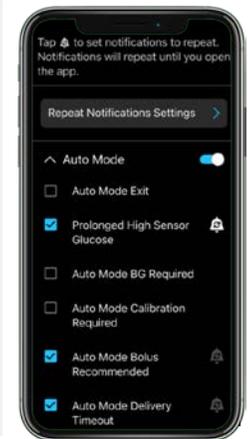

ALWAYS FOLLOW THE DIRECTIONS FOR USE