

# GUARDIAN™ CONNECT TRAINING CHECKLIST

Medtronic

Patient Name	
Patient Phone Number	
Guardian Connect Transmitter Serial Number	
Blood Glucose Meter Brand	
Product Trainer	
Prescribing Physician	
Training Date	

- Reviewed Patient getting Started Guide  Yes  No
- Reviewed Getting started with Medtronic CareLink® Personal  Yes  No
- Set up CareLink Personal account if required  Yes  No
- Downloaded Guardian Connect Application from Apple App Store  Yes  No

## Keys to success: sensor glucose ≠ blood glucose

*Patient has verbalised understanding of:*

- Difference between sensor glucose (SG) and blood glucose (BG)
- BG confirmations required for treatment decisions

Comments:

---

---

## Key to success: sensor insertion

*Patient has verbalised understanding and demonstrated the following:*

- Site selection, rotation and preparation  Correct steps to sensor insertion
- Use of sarter including 5 second hold  Applying pressure to sensor adhesive for several seconds

Comments:

---

---

## Keys to success: taping

Patient has verbalised and demonstrated the following:

- |  |  |
|--|--|
| <input type="checkbox"/> Importance of applying overtape                   | <input type="checkbox"/> Steps for applying overtape correctly |
| <input type="checkbox"/> Applying pressure to overtape for several seconds | <input type="checkbox"/> Connecting transmitter to sensor      |
| <input type="checkbox"/> Applying adhesive tab carefully onto transmitter  |  |

Comments:

---

---

## Initialising the sensor:

Patient has verbalised understanding of:

- |  |  |
|--|--|
| <input type="checkbox"/> Steps to start new sensor           | <input type="checkbox"/> Following steps on app screen |
| <input type="checkbox"/> Initialisation period               | <input type="checkbox"/> Reading the sensor graphs     |
| <input type="checkbox"/> Reading sensor status screen on app |  |

Comments:

---

---

## Key to success: personalise alarms and alerts

- |  |  |
|--|--|
| <input type="checkbox"/> Importance of Personalising Alerts      | <input type="checkbox"/> Low and High Glucose alerts           |
| <input type="checkbox"/> Predictive Alert: Low and High settings | <input type="checkbox"/> Rate Alert: Rise Alert and Fall Alert |
| <input type="checkbox"/> Calibration reminder                    | <input type="checkbox"/> Setting Snooze Times                  |

Comments:

---

---

## Keys to success: trends

Patient has verbalised understanding of:

- Focusing on Sensor Glucose trends

Comments:

---

---

## Keys to success: Calibration

Patient has verbalised understanding of:

- Importance of calibration
- Optimal times to calibrate
- Manually calibrating

Comments: \_\_\_\_\_

## Keys to success: Medtronic Carelink® Personal

- Instructed on Medtronic CareLink set-up if not done
- How to set up care partners and share information
- How to set up alerts for care partners

Comments: \_\_\_\_\_

## Entering event markers

- Understands how to enter appropriate event marker
- Demonstrates how to identify event markers on home screen

Comments: \_\_\_\_\_

## Additional topics have been discussed:

- Charging the Transmitter
- Removal for Xray, CT Scan, MRI
- App must stay running in background to receive alerts with "do not disturb" off
- Calibration reminder

## The following items have been completed:

- Settings entered in consultation with trainer/HCP
- Follow-up plan outlined and reviewed
- Verified all settings entered correctly

## Discussed when to contact

- Healthcare Professional
- Medtronic Diabetes Helpline:  
1 800 777 808 (Toll Free in Australia)

Patient Signature: \_\_\_\_\_ Date: \_\_\_\_\_

Trainer Signature: \_\_\_\_\_ Date: \_\_\_\_\_

Your personal information collected here is used by Medtronic Australasia Pty Ltd to record and assist you with your training requirements, in accordance with our Privacy Policy (see [www.medtronic.com.au](http://www.medtronic.com.au)). Medtronic and Medtronic CareLink are registered trademarks and Guardian is a trademark of Medtronic MiniMed, Inc. ©2017 Medtronic Australasia Pty Ltd. All rights reserved.