




Thursday 07-26

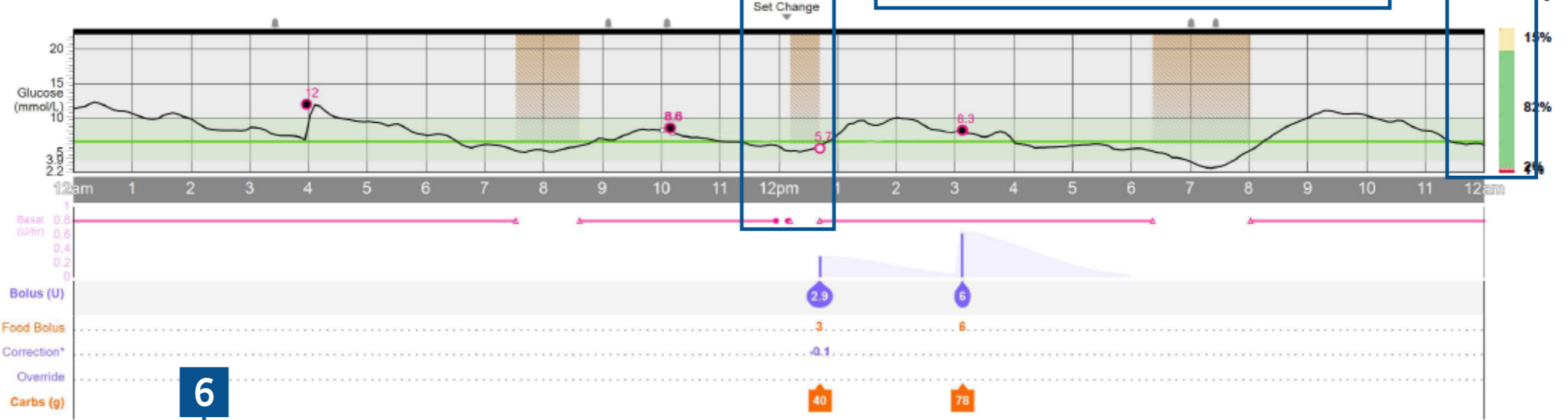

Friday 07-27

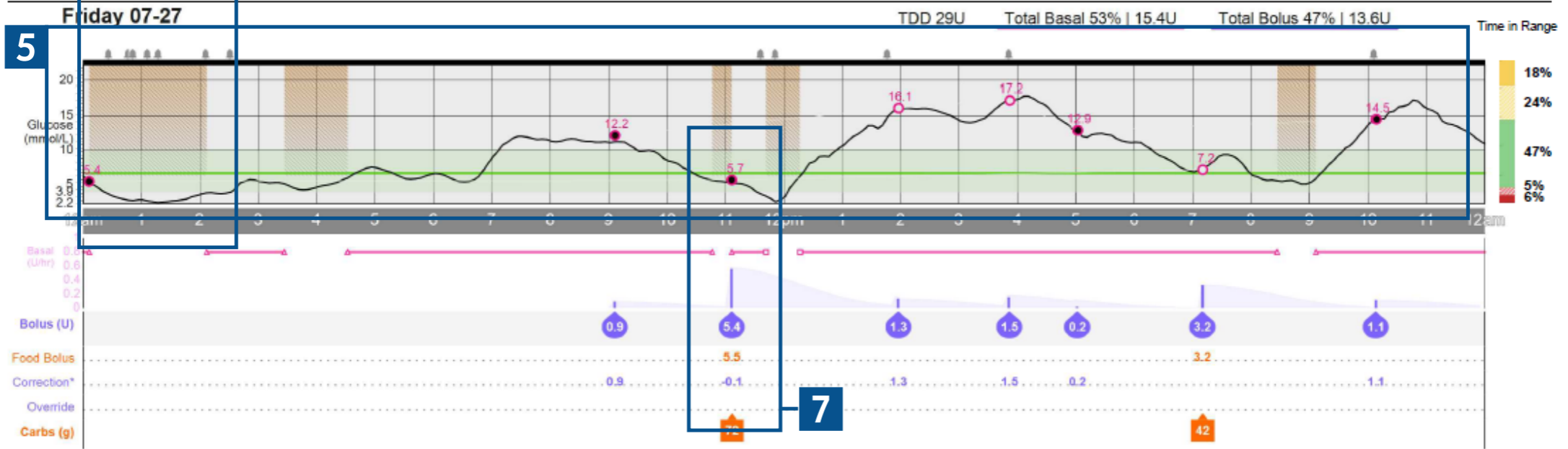

*Active insulin considered

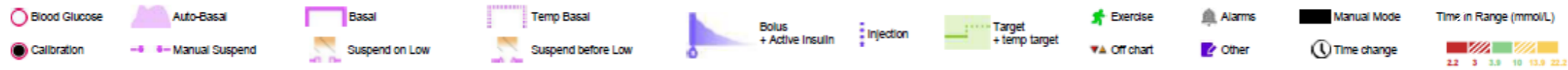

Example of real patient data


Thursday 07-26

TDD 25.4U

Total Basal 65% | 16.5U

Total Bolus 35% | 8.9U

Time in Range

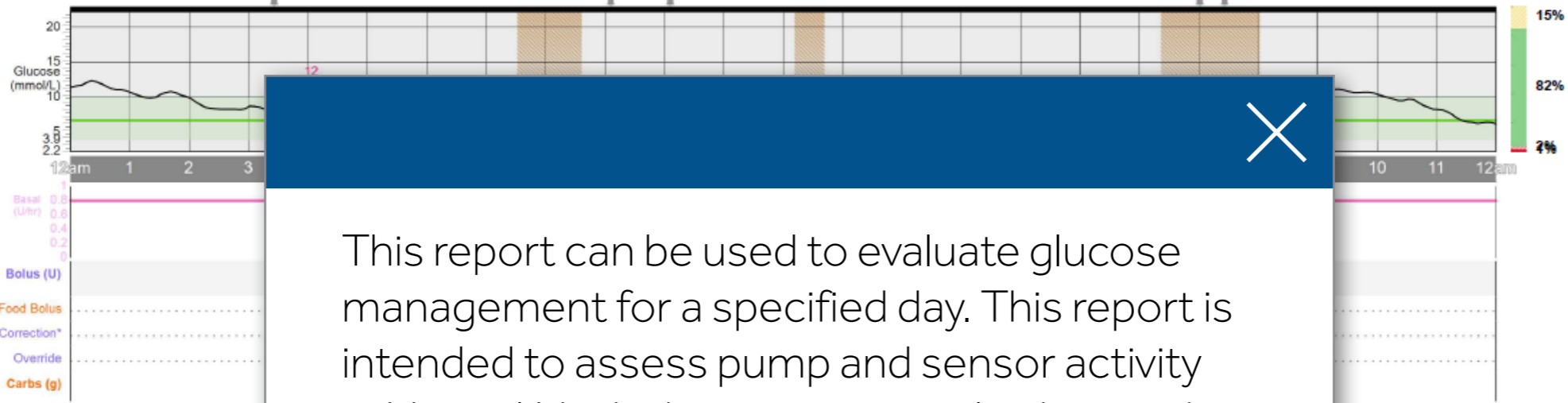


This report can be used to evaluate glucose management for a specified day. This report is intended to assess pump and sensor activity in Manual Mode, however it can also be used to view days in Auto Mode.


Friday 07-27

TDD 25.4U

Time in Range

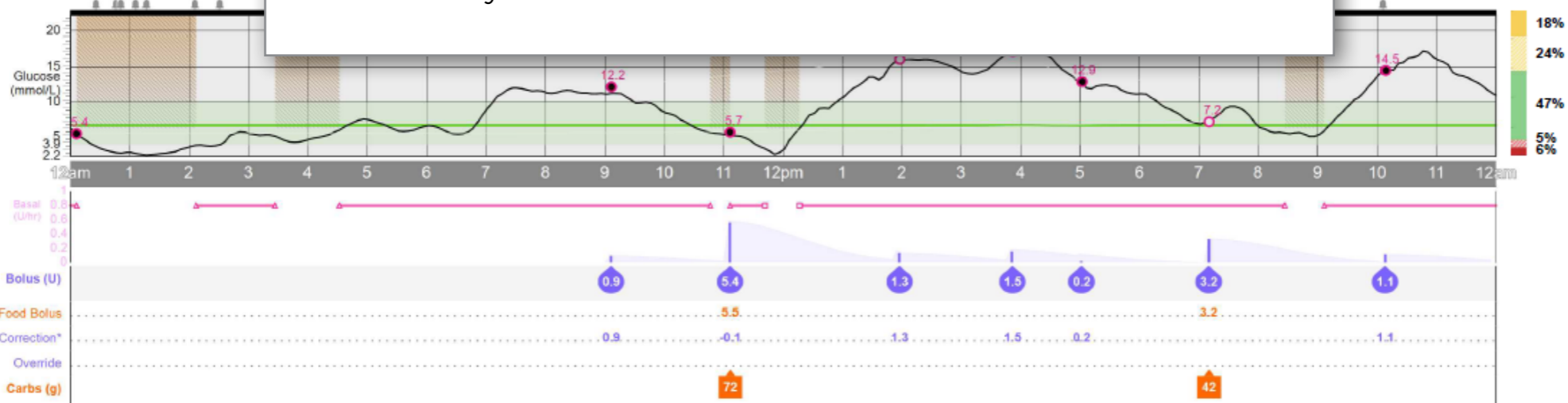


*Active insulin considered

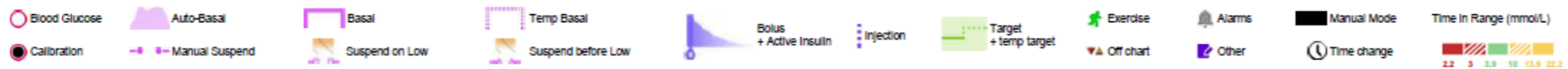

2

Thursday 07-26


TDD 25.4U Total Basal 65% | 16.5U Total Bolus 35% | 8.9U

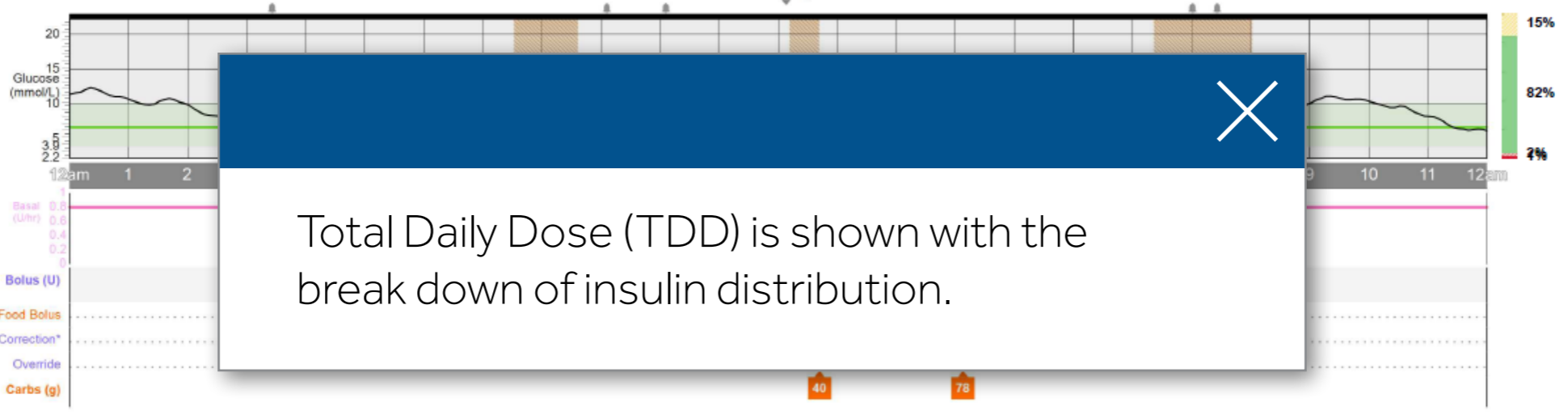


Total Daily Dose (TDD) is shown with the break down of insulin distribution.

Friday 07-27

TDD 29U Total Basal 53% | 15.4U Total Bolus 47% | 13.6U

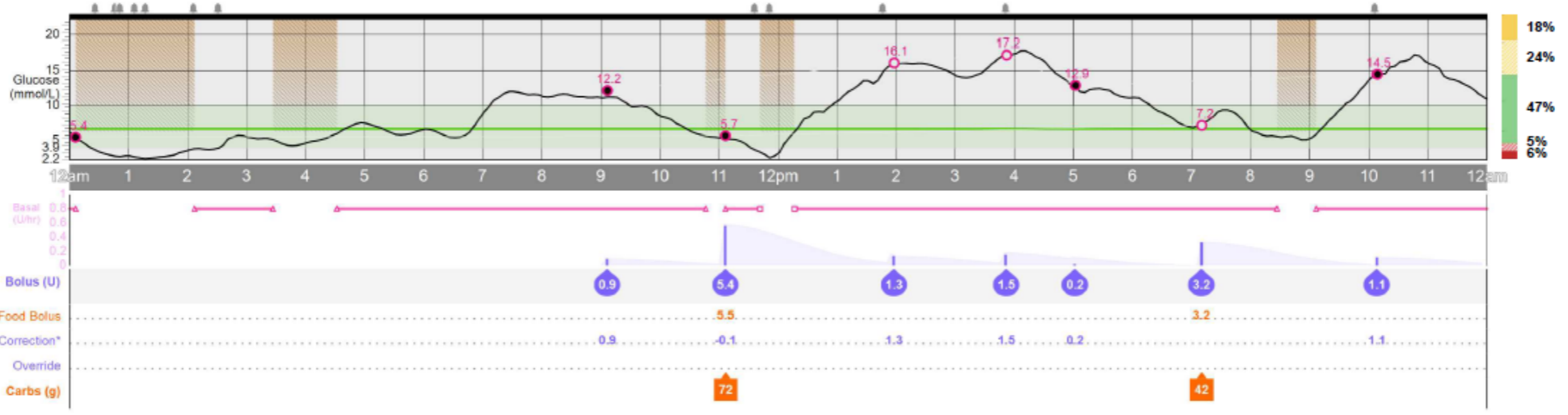

*Active insulin considered

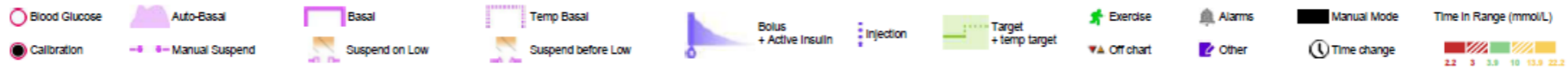


Thursday 07-26

TDD 25.4U

Total Basal 65% | 16.5U

Total Bolus 35% | 8.9U

3
Time in Range

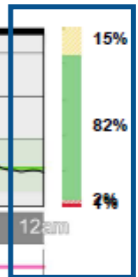


Friday 07-27

TDD 29U


Total Basal 53% | 15.4U

Total Bolus 47% | 13.6U

Time in Range

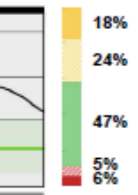

*Active insulin considered

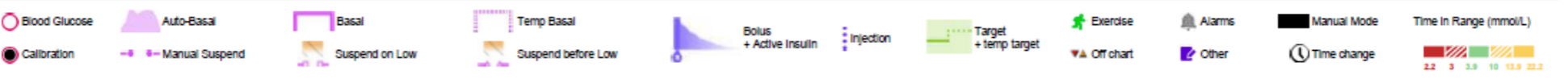


Thursday 07-26

TDD 25.4U

Total Basal 65% | 16.5U

Total Bolus 35% | 8.9U

Time in Range

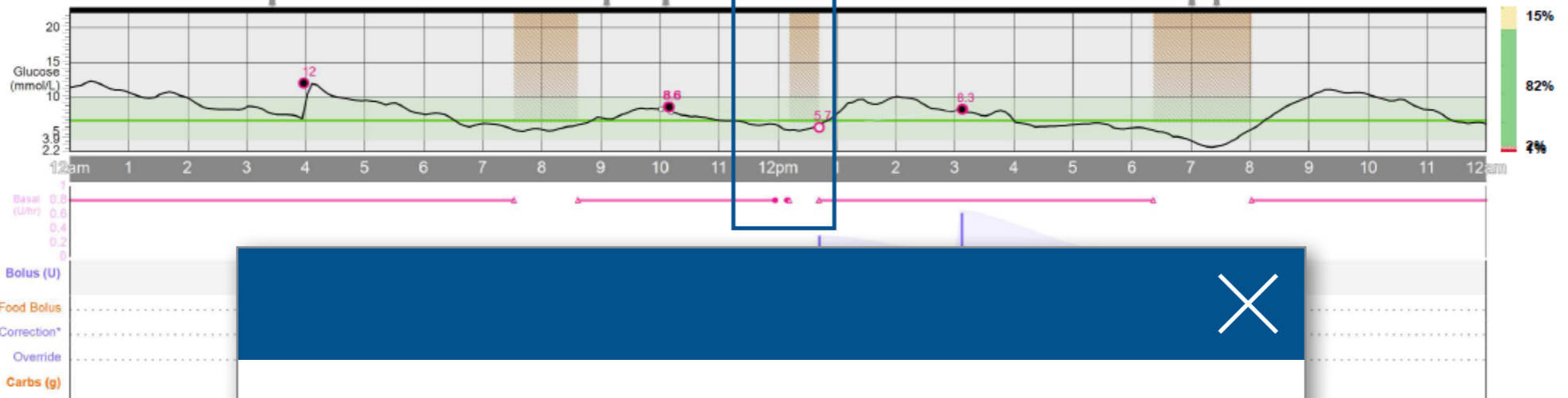

During this time of the day, the patient suspended the pump to perform an infusion set change.


Friday 07-27

TDD 25.4U


Total Basal 65% | 16.5U

Total Bolus 35% | 8.9U

Time in Range

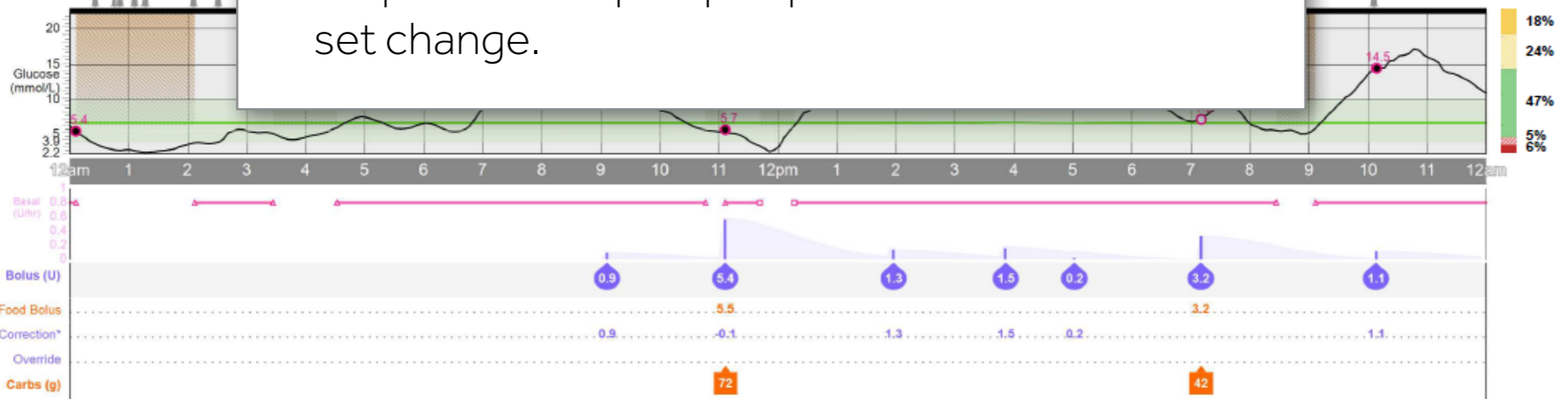

*Active insulin considered

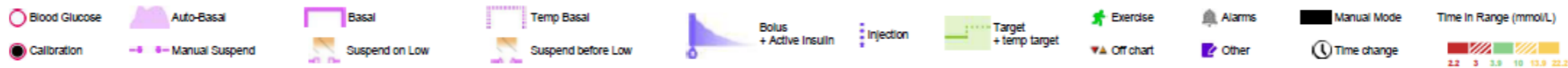


Thursday 07-26

TDD 25.4U

Total Basal 65% | 16.5U

Total Bolus 35% | 8.9U

Time in Range

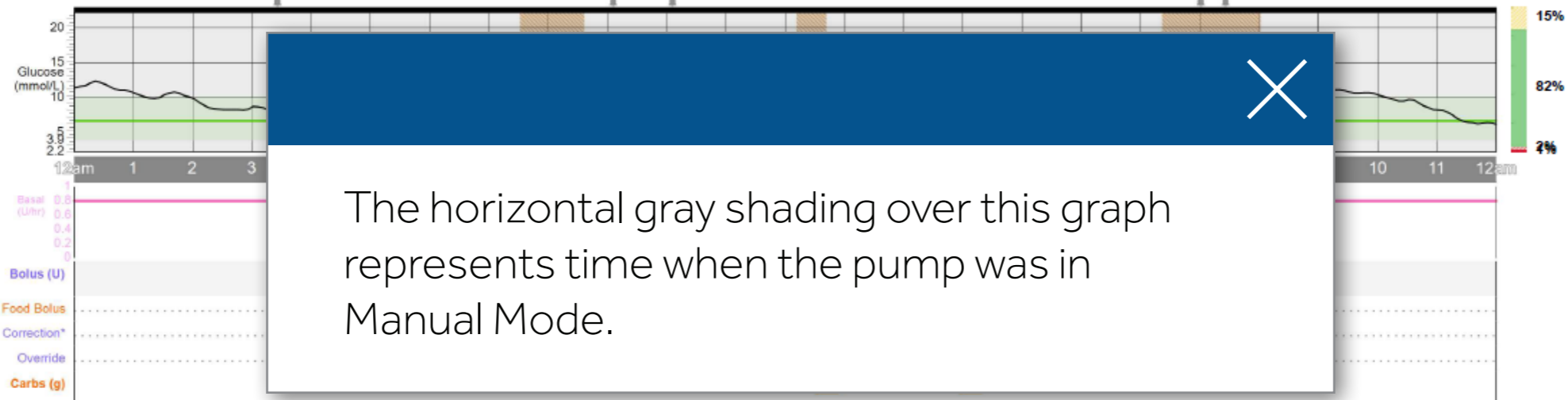

Friday 07-27

TDD 29U


Total Basal 53% | 15.4U

Total Bolus 47% | 13.6U

Time in Range



*Active insulin considered

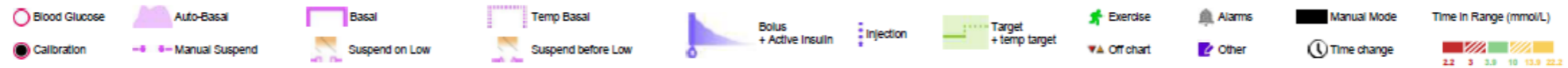


Thursday 07-26

TDD 25.4U

Total Basal 65% | 16.5U

Total Bolus 35% | 8.9U

Time in Range

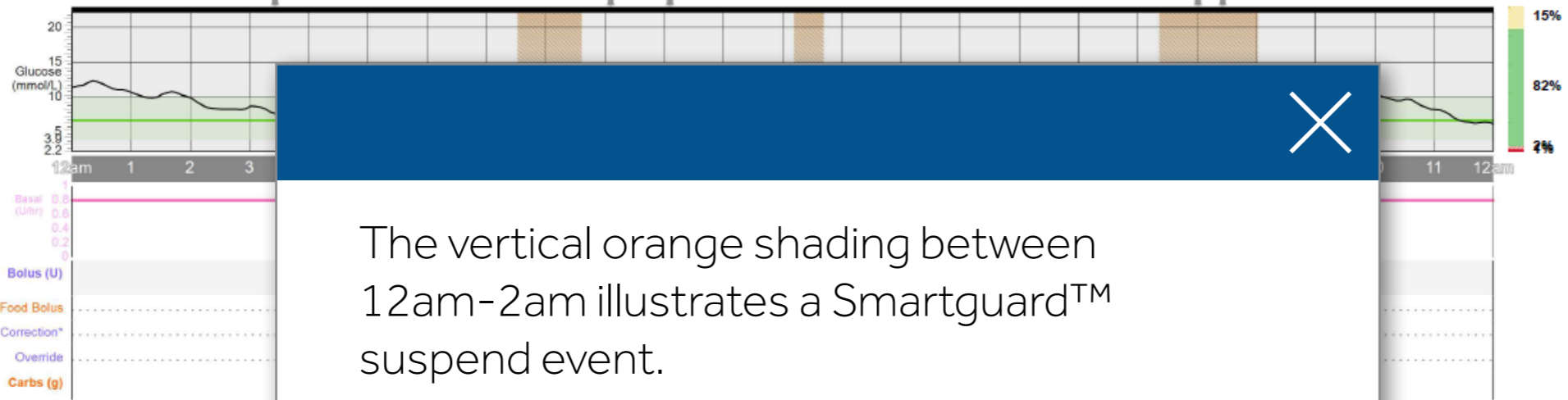

The vertical orange shading between 12am-2am illustrates a Smartguard™ suspend event.

6


Friday 07-27

TDD 29U


Total Basal 53% | 15.4U


Total Bolus 47% | 13.6U

Time in Range

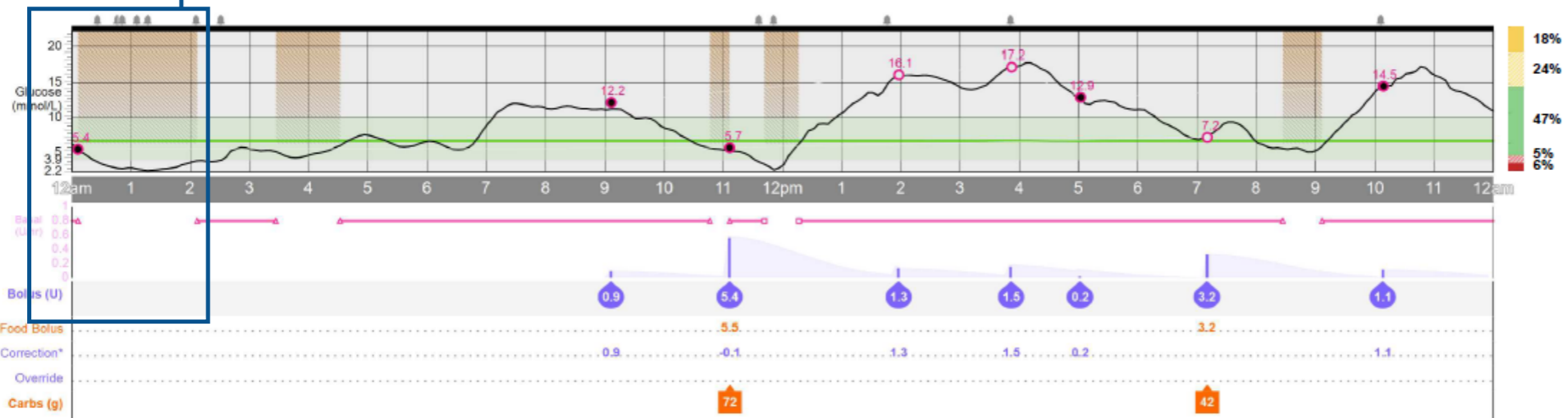

*Active insulin considered

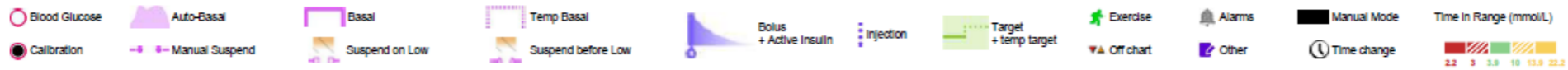

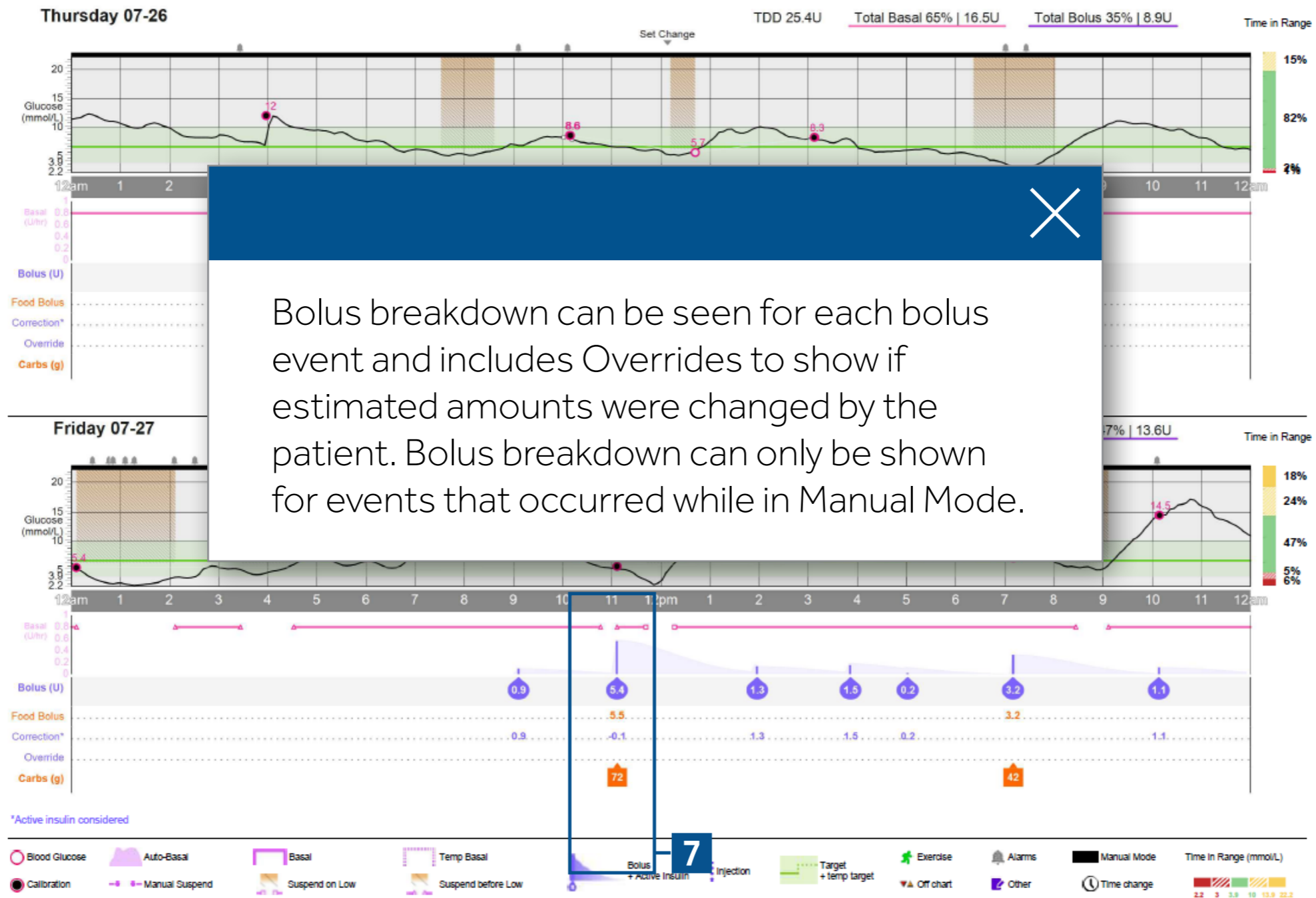

Example of real patient data

Medtronic

Medtronic Australasia Pty Ltd

2 Alma Road

Macquarie Park , NSW 2113

www.medtronic-diabetes.au

1800 777 808

© 2019 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further. Together are trademarks of Medtronic.
All other brands are trademarks of a Medtronic company.
approved 5945032019