

MINIMED™ 770G AUTO MODE TRAINING CHECKLIST

PATIENT INFORMATION

| | | | |
|-------|----------------------------|--------------|----------------|
| NAME: | CERTIFIED PRODUCT TRAINER: | CLINIC NAME: | TRAINING DATE: |
|-------|----------------------------|--------------|----------------|

DEVICE SERIAL NUMBERS:

KNOWLEDGE CHECK (understanding key terms and concepts)

- | | |
|--|---|
| <input type="checkbox"/> Completed MyLearning <input type="checkbox"/> Auto Mode Getting Started Guide received <input type="checkbox"/> CareLink™ Personal set up | <input type="checkbox"/> Discuss expectations of Auto Mode compared to current therapy <input type="checkbox"/> Medtronic apps installed |
|--|---|

ENSURE AUTO MODE READINESS

- | | |
|--|---|
| <input type="checkbox"/> Auto Mode warm-up is complete <input type="checkbox"/> Sensor inserted and working | <input type="checkbox"/> Review pump settings <input type="checkbox"/> Confirm no active temp basal, manual suspend or dual/square bolus |
|--|---|

AUTO MODE START (understanding key terms and concepts)

- | | |
|---|--|
| <input type="checkbox"/> Discuss basic concept of Auto Mode <input type="checkbox"/> Auto Mode turned on <input type="checkbox"/> Auto Mode BG alert turned on <input type="checkbox"/> Review Auto Mode Readiness screen and icons <input type="checkbox"/> Enter BG if required | <input type="checkbox"/> Review Auto Mode home screen (blue shield) <input type="checkbox"/> Review Auto Mode sensor graphs (pink dots) <input type="checkbox"/> Discuss Temp Target <input type="checkbox"/> How to calibrate and bolus in Auto Mode <input type="checkbox"/> Review Safe Basal (grey shield) |
|---|--|

KEYS TO SUCCESS IN AUTO MODE (understanding key terms and concepts)

- | | |
|--|---|
| <input type="checkbox"/> Bolus before meals <input type="checkbox"/> Calibrate 3-4 times per day at stable times <input type="checkbox"/> Read and respond to pump messages <input type="checkbox"/> Enter BG if requested by pump <input type="checkbox"/> Give correction bolus if recommended by pump | Auto Mode Exits <input type="checkbox"/> Check Auto Mode Readiness screen for re-entry information <input type="checkbox"/> Re-enter as soon as possible (look for blue shield) <input type="checkbox"/> Review how to turn SmartGuard™ low settings on if planning on remaining in Manual Mode |
|--|---|

NOTES / FOLLOW UP APPOINTMENT SCHEDULE

Patient signature: _____ Date: _____

Trainer signature: _____ Date: _____