



MiniMed™ 780G System changing the MiniMed™ Mio™ infusion set quick reference guide

Start here

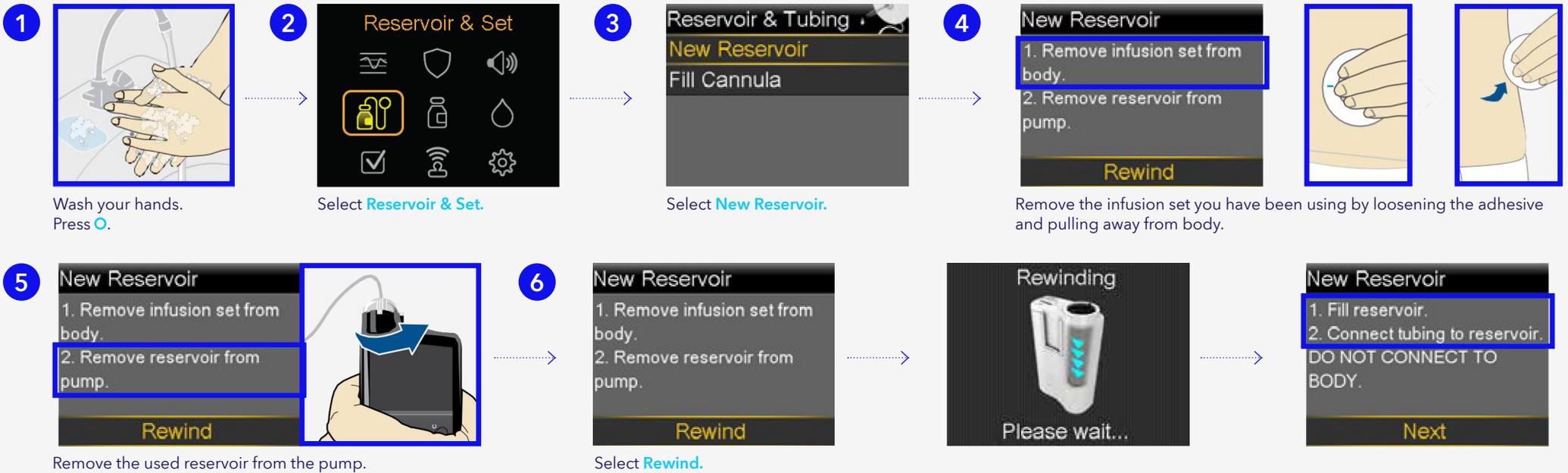


Fill reservoir & connect to the infusion set tubing

Follow the next steps to fill reservoir with insulin and connect to the infusion set tubing.

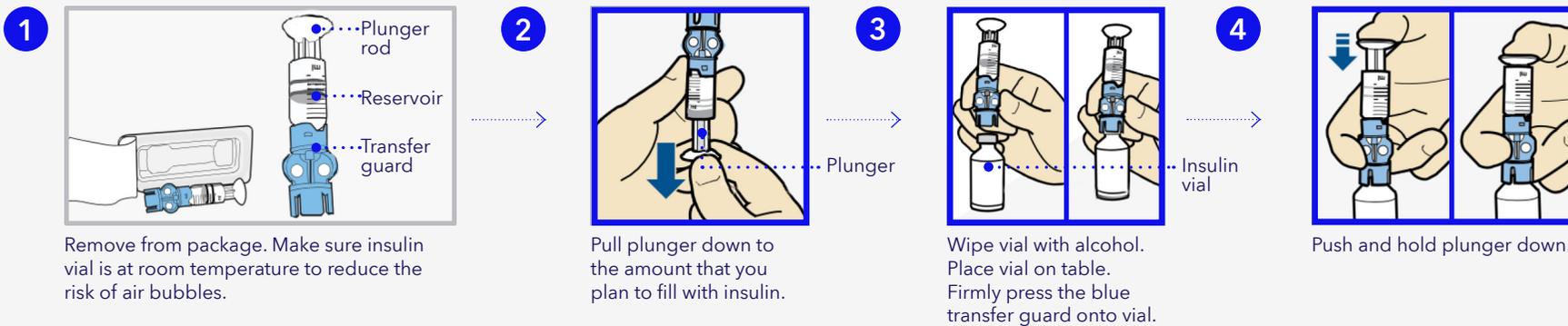


MiniMed™ 780G System changing the MiniMed™ Mio™ infusion set quick reference guide


 If insulin or any liquid gets inside the tubing connector, it can temporarily block the vents that allow the pump to properly fill the infusion set. This may result in the delivery of too little or too much insulin, which could cause hyperglycaemia or hypoglycaemia.

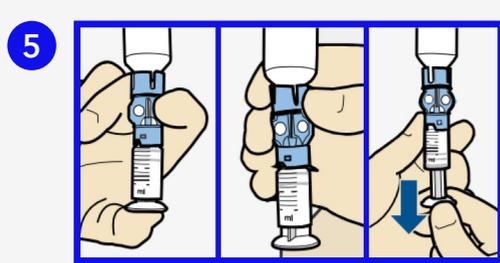


5 With your thumb still on the plunger, flip over so vial is on top. Release thumb and pull plunger down to fill with insulin.

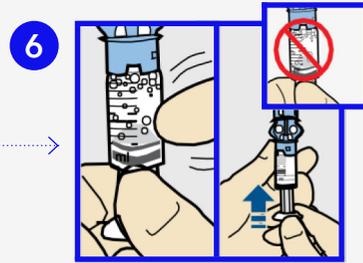

6 Tap the reservoir to move air bubbles to top of reservoir. Push plunger up to move air into vial.

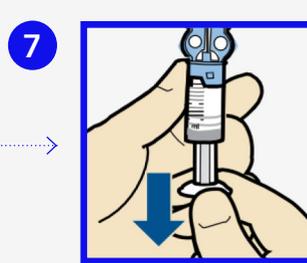


7 If needed, pull plunger back down to amount of insulin needed for 2-3 days.

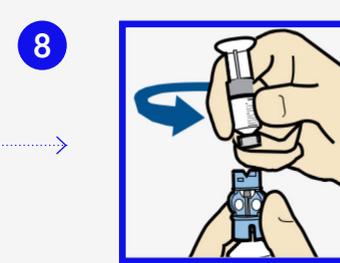


8 To avoid getting insulin on the top of the reservoir, turn vial over so it is upright. Hold transfer guard and turn reservoir counter-clockwise and remove from transfer guard.

Connect reservoir to infusion set


You will place the tubing connector onto the end of the infusion set to the filled reservoir.

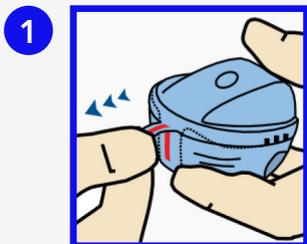


1 Remove the infusion set from packaging by pulling down the red plastic tab.

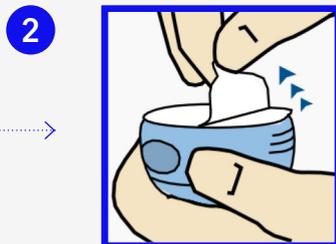


2 Pull the paper tab to remove the sterile paper backing from the bottom.

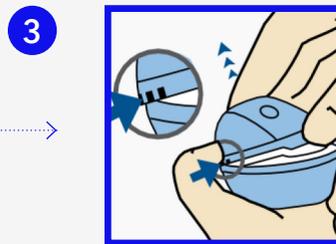


3 Squeeze the three raised markings on both sides of the lid. Lift and remove.

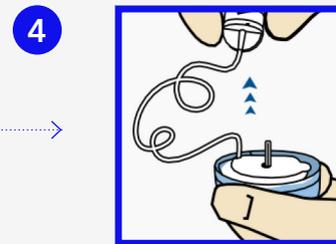


4 Free tubing from slot. Gently unwind tubing in a counter clockwise direction.

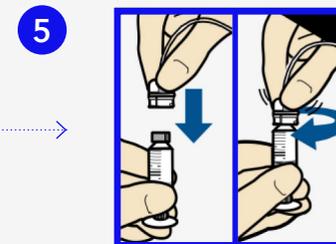

5 Gently push the tubing connector onto reservoir. Turn clockwise until locked. You will hear a click.

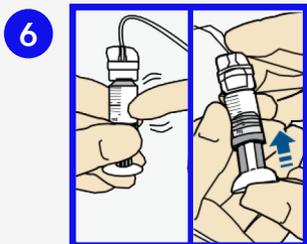

6 If you see air bubbles, tap reservoir to move them to top. Push plunger just a bit to move them into tubing.

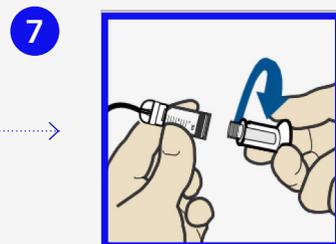

7 Twist the plunger counter-clockwise to loosen and remove.

The backlight may have turned off


Press any button to turn the screen back on.

1  **2**  **3** 

Press **O** to open the menu. Select . Select **Next**.

Place reservoir into pump



Now place the filled reservoir into the reservoir compartment of the pump.

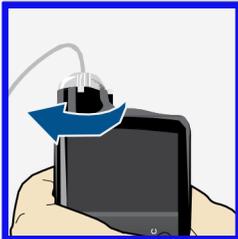1   **2**  **3** 

Place reservoir into pump. Turn clockwise until you feel reservoir lock into place. Select **Next**.

Load reservoir and fill tubing

Follow these steps to load the reservoir and fill the tubing.

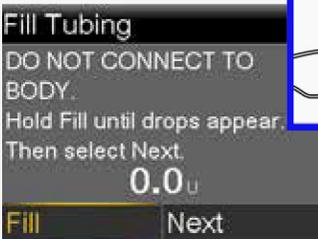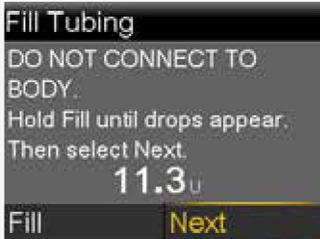1  **2**  **3**   **4** 

Select **Load** and keep holding **O**. When you see this screen, select **Next**. Select **Fill** and keep holding **O** until you see drops at the end of tubing, then let go. Drops at end of tubing. After you see drops, press **>** and select **Next**.

MiniMed™ 780G System changing the MiniMed™ Mio™ infusion set quick reference guide


Inert infusion set

Next, follow these steps to insert the infusion set into your body.


Fill Cannula?

1. Insert infusion set into body.
2. Select Fill to fill cannula or Done if not needed.


Fill Done

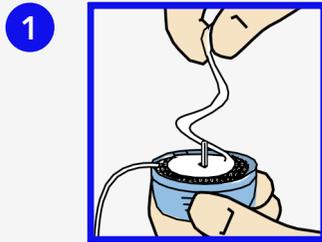


1 Gently peel paper to expose the adhesive.

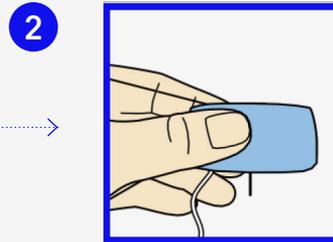


2 Turn over and hold by the lined ridges on the sides.

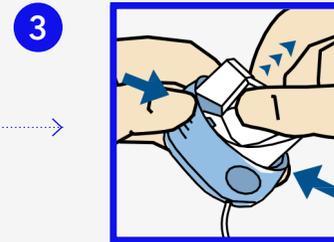

3 With other hand pull upon the center handle until it clicks and locks into place.

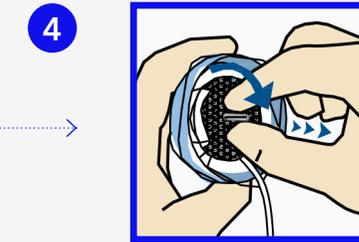


4 Carefully hold the needle guard by tip. Gently turn the needle guard and pull to remove it.

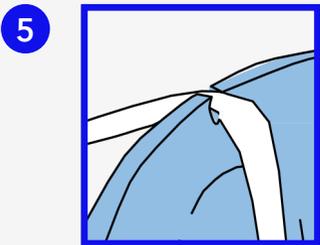


5 Place the tubing in slot on side of serter.


CAUTION:
Be careful not to pull the tubing tightly when placing it in the slot.

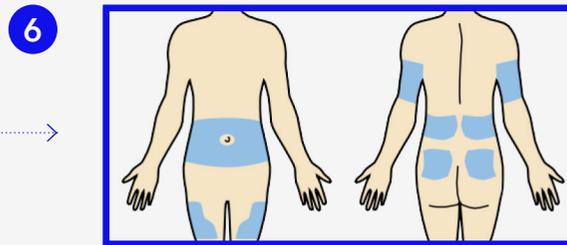


6 Choose an insertion site from the shaded areas shown here. Wipe with alcohol or other antiseptic.

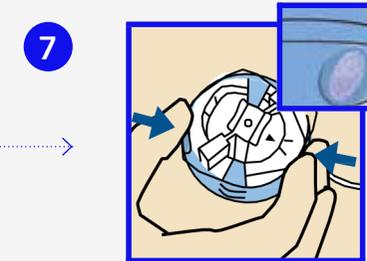

7 Place against prepared site on body. To insert, press the round indentations on each side of serter to insert needle.

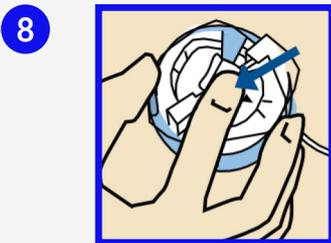

8 Push down on center of serter to press adhesive against skin.

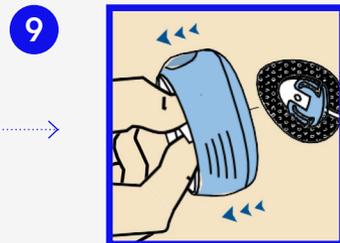

9 To remove serter, hold the center handle and gently pull it straight out away from body. Press the adhesive securely against skin.




10 Place the lid back onto serter to safely discard needle.

MiniMed™ 780G System changing the MiniMed™ Mio™ infusion set quick reference guide

Fill cannula

You will now fill the cannula, the little tube under your skin, with insulin.

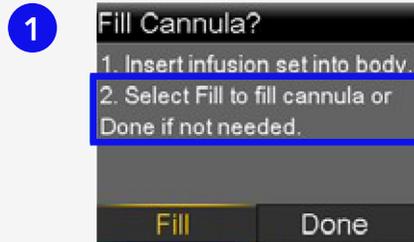


Select **Fill**.




Select **Fill amount** and enter:

- ⌂ 0.300 if using 6mm cannula
- ⌂ 0.500 if using 9mm cannula and press **O**.

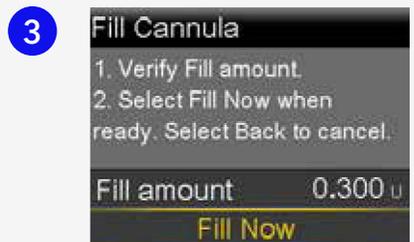

Select **Fill Now**.

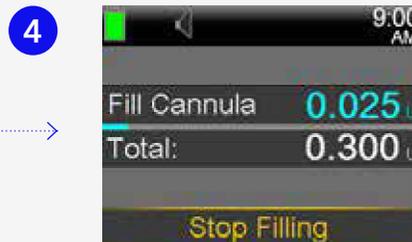

The Home screen displays the insulin as it fills the cannula.



NOTE:

Your pump will remember the **Fill amount** that you used last. Always verify that the Fill amount is correct.

- ⌂ If it is correct, press **✓** to **Fill Now** and press **O**.
- ⌂ If it is incorrect, press **O**. Change to correct amount and press **O**. Select **Fill Now**.



NOTE:

Select **Stop Filling** if you need to stop, for example, if you notice the **Total** amount is incorrect. This should rarely happen if you have verified the **Fill amount** on the previous screen.

Your infusion set change is now complete!

Medtronic

Medtronic Australasia Pty Ltd
2 Alma Road
Macquarie Park, NSW 2113, Australia

Tel: +61 2 9857 9000
Fax: +61 2 9889 5167
Toll Free: 1800 777 808

E-Mail: australia.diabetes@medtronic.com
Facebook: www.facebook.com/MedtronicDiabetesAUS
YouTube: [MedtronicDiabetesANZ](https://www.youtube.com/MedtronicDiabetesANZ)

<https://www.medtronic-diabetes.com.au/>

Always read the instructions for use.

For detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events, please consult the device manual. For further information, contact your local Medtronic representative.

© 2021 Medtronic Australasia Pty Ltd. All Rights Reserved. Please note: In contacting the Diabetes Toll Free, your personal and health information may be disclosed to an operator located outside Australia. 10228-102021