



MiniMed™ 780G system changing the quick-set infusion set quick reference guide

Start here

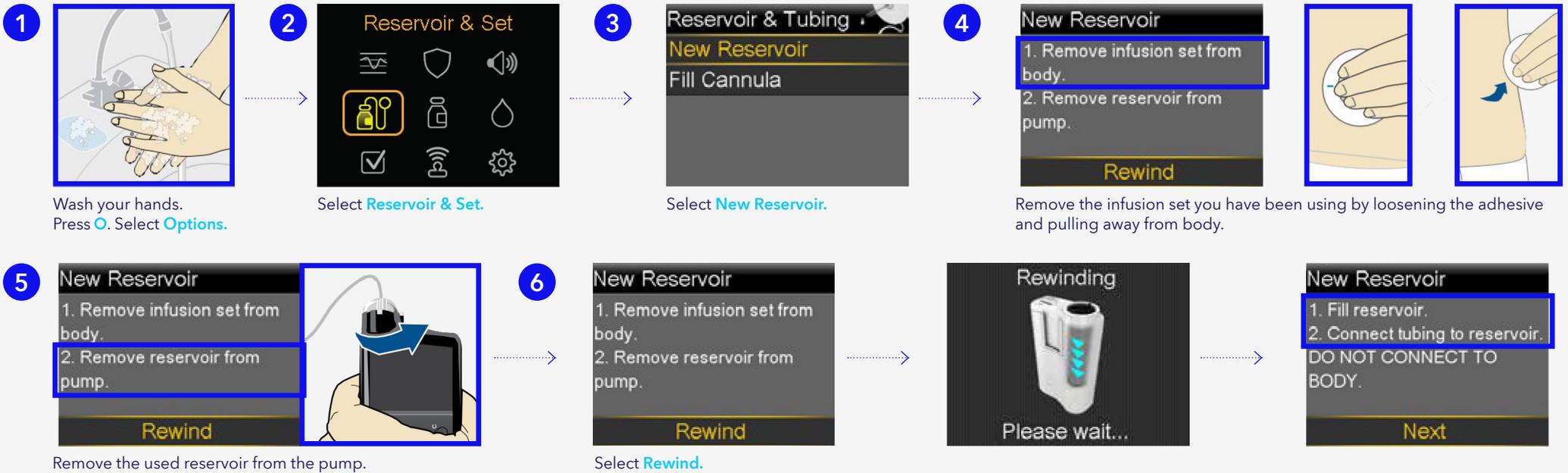


Fill reservoir & connect to the infusion set tubing

Follow the next steps to fill reservoir with insulin and connect to the infusion set tubing.

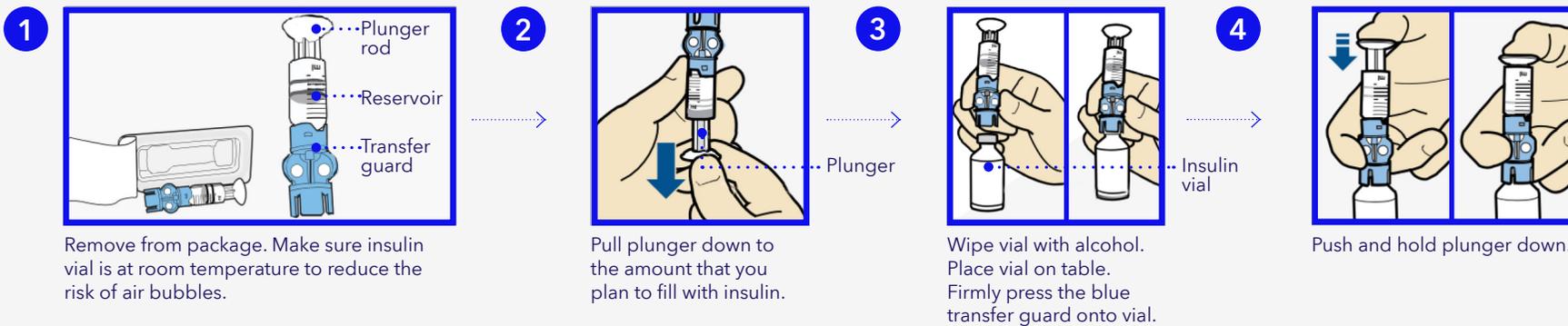


MiniMed™ 780G system changing the quick-set infusion set quick reference guide


! If insulin or any liquid gets inside the tubing connector, it can temporarily block the vents that allow the pump to properly fill the infusion set. This may result in the delivery of too little or too much insulin, which could cause hyperglycaemia or hypoglycaemia.

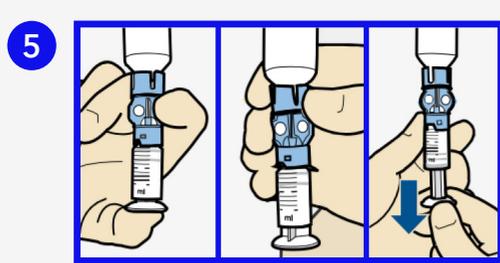


With your thumb still on the plunger, flip over so vial is on top. Release thumb and pull plunger down to fill with insulin.

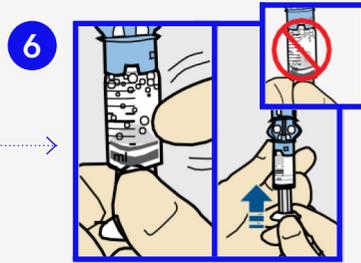

Tap the reservoir to move air bubbles to top of reservoir. Push plunger up to move air into vial.

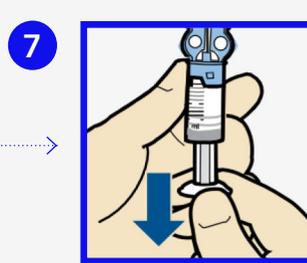


If needed, pull plunger back down to amount of insulin needed for 2-3 days.

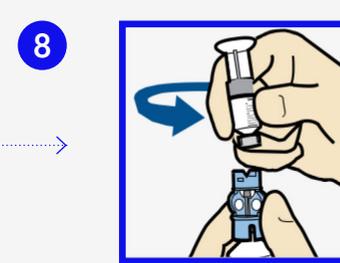


To avoid getting insulin on the top of the reservoir, turn vial over so it is upright. Hold transfer guard and turn reservoir counter-clockwise and remove from transfer guard.

Connect reservoir to infusion set


You will place the tubing connector onto the end of the infusion set to the filled reservoir.

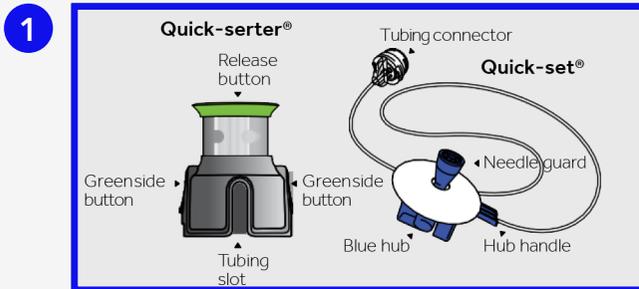


Remove infusion set from package. Remove the paper that holds the tubing together.

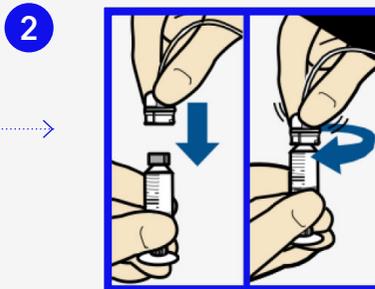

Gently push the tubing connector onto reservoir. Turn clockwise until locked. You will hear a click.

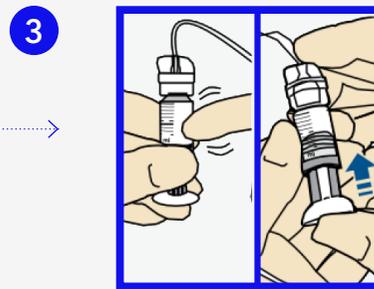

If you see air bubbles, tap reservoir to move them to top. Push plunger just a bit to move them into tubing.

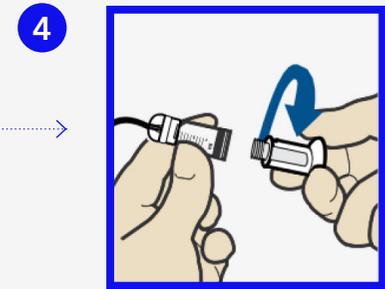

Twist the plunger counter-clockwise to loosen and remove.

The backlight may have turned off


Press any button to turn the screen back on.



Press **O** to open the menu.



Select

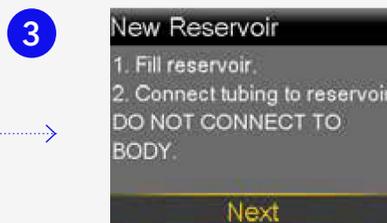

Select **Next**.

MiniMed™ 780G system changing the quick-set infusion set quick reference guide


Place reservoir into pump

Now place the filled reservoir into the reservoir compartment of the pump.

1 **New Reservoir**
3. Place reservoir into pump and lock.
DO NOT CONNECT TO BODY.
Next



2

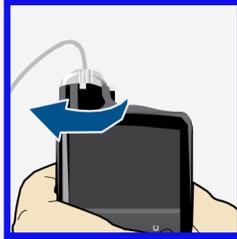

3 **New Reservoir**
3. Place reservoir into pump and lock.
DO NOT CONNECT TO BODY.
Next

Place reservoir into pump.

Turn clockwise until you feel reservoir lock into place.

Select **Next**.

Load reservoir and fill tubing

Follow these steps to load the reservoir and fill the tubing.

1 **Load Reservoir**
Select Load and hold until complete.
DO NOT CONNECT TO BODY.
Load **Next**



2 **Load Reservoir**
Complete
DO NOT CONNECT TO BODY.
Load **Next**



3 **Fill Tubing**
DO NOT CONNECT TO BODY.
Hold Fill until drops appear.
Then select Next.
0.0 u
Fill **Next**



Drops at end of tubing

4 **Fill Tubing**
DO NOT CONNECT TO BODY.
Hold Fill until drops appear.
Then select Next.
11.3 u
Fill **Next**

Select **Load** and keep holding **O**.

When you see this screen, select **Next**.

Select **Fill** and keep holding **O** until you see drops at the end of tubing, then let go.


After you see drops, press **>** and select **Next**.

Insert infusion set

Next, follow these steps to insert the infusion set into your body.


Fill Cannula?
1. Insert infusion set into body.
2. Select Fill to fill cannula or Done if not needed.
Fill **Done**

1

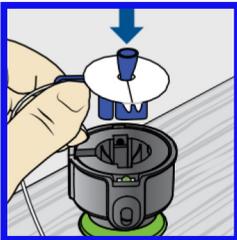

Place the MiniMed Quick-serter insertion device onto a sturdy flat surface with handle facing down. Place **blue** hub into serter, placing the handle in the tubing slot.

2

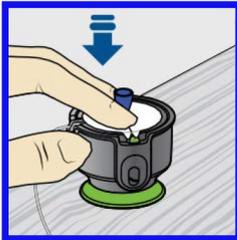

Use two fingers to seat the infusion set inside the serter securely and gently push down.

3



Peel the paper from the adhesive on both sides of the needle guard.

Insert infusion set


Next, follow these steps to insert the infusion set into your body.

4




Pull green handle down until it clicks. The two green side buttons will also click into place. Do not press the green side buttons at this time.

5

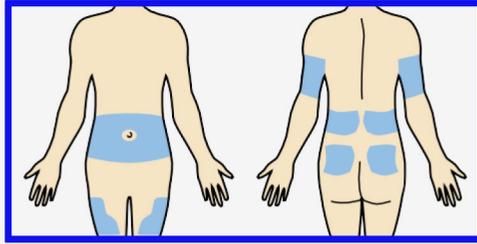

Choose an insertion site from the shaded areas shown here. Wipe with alcohol or other antiseptic.

6

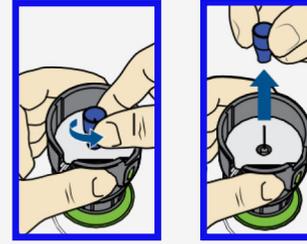


Turn to loosen needle guard and pull.

7




Hold serter against cleaned site.

8

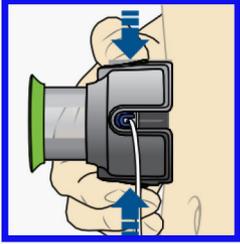


Press the two green side buttons at the same time. If they are not pressed at the same time the infusion set will not insert properly.

9

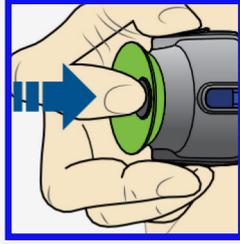


Press down the release button of the serter to release the infusion set.

10

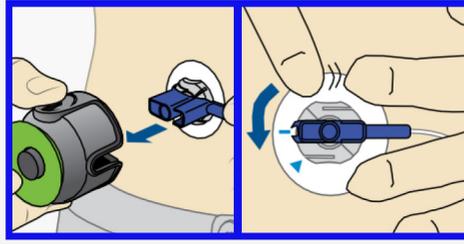


Pull serter away from body. Press adhesive against skin.

11

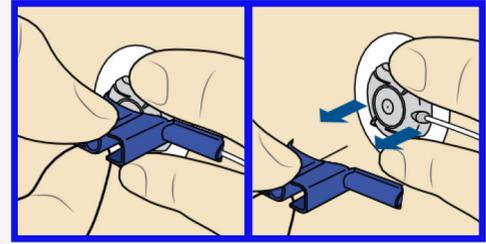

Hold infusion set. Pull blue handle straight out to remove needle.

12

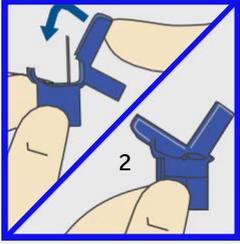

Fold blue handle until locked.




Action:
Dispose blue handle into a sharps container.


MiniMed™ 780G system changing the quick-set infusion set quick reference guide

Fill cannula

You will now fill the cannula, the little tube under your skin, with insulin.

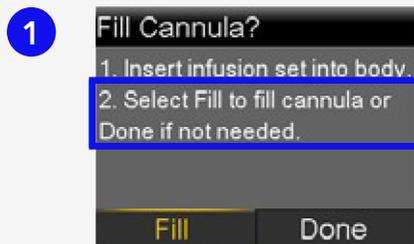

Select **Fill**.

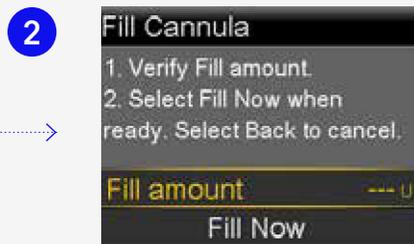

Select **Fill amount** and enter:

- ⌂ 0.300 if using 6mm cannula
- ⌂ 0.500 if using 9mm cannula and press **⊙**.



Select **Fill Now**.



The Home screen displays the insulin as it fills the cannula.



Note:
Your pump will remember the **Fill amount** that you used last. Always verify that the Fill amount is correct.

- ⌂ If it is correct, press **▼** to **Fill Now** and press **⊙**.
- ⌂ If it is incorrect, press **⊙**. Change to correct amount and press **⊙**. Select **Fill Now**.



Note:
Select **Stop Filling** if you need to stop, for example, if you notice the **Total** amount is incorrect. This should rarely happen if you have verified the **Fill amount** on the previous screen.

Your infusion set change is now complete!

Medtronic

Medtronic Australasia Pty Ltd
2 Alma Road
Macquarie Park, NSW 2113, Australia

Tel: +61 2 9857 9000
Fax: +61 2 9889 5167
Toll Free: 1800 777 808

E-Mail: australia.diabetes@medtronic.com
Facebook: www.facebook.com/MedtronicDiabetesAUS
YouTube: [MedtronicDiabetesANZ](https://www.youtube.com/MedtronicDiabetesANZ)

<https://www.medtronic-diabetes.com.au/>

Always read the instructions for use.

For detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events, please consult the device manual. For further information, contact your local Medtronic representative.

© 2021 Medtronic Australasia Pty Ltd. All Rights Reserved. Please note: In contacting the Diabetes Toll Free, your personal and health information may be disclosed to an operator located outside Australia. 10317-102021