



MiniMed™ 780G System changing the MiniMed™ Sure-T® infusion set quick reference guide

Start here

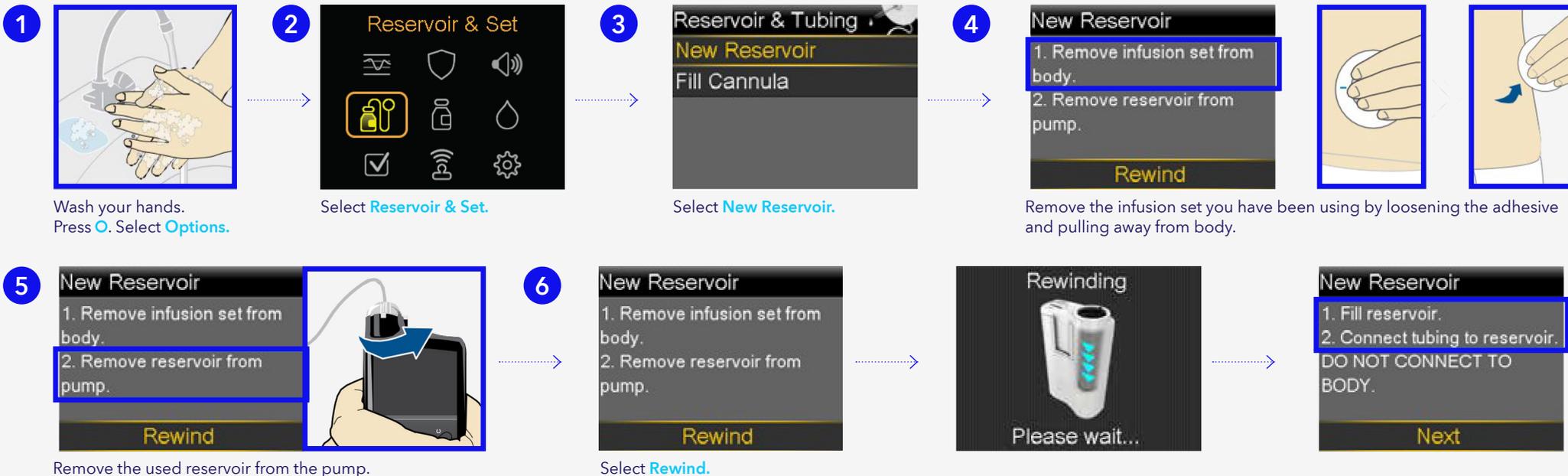


Fill reservoir & connect to the infusion set tubing

Follow the next steps to fill reservoir with insulin and connect to the infusion set tubing.

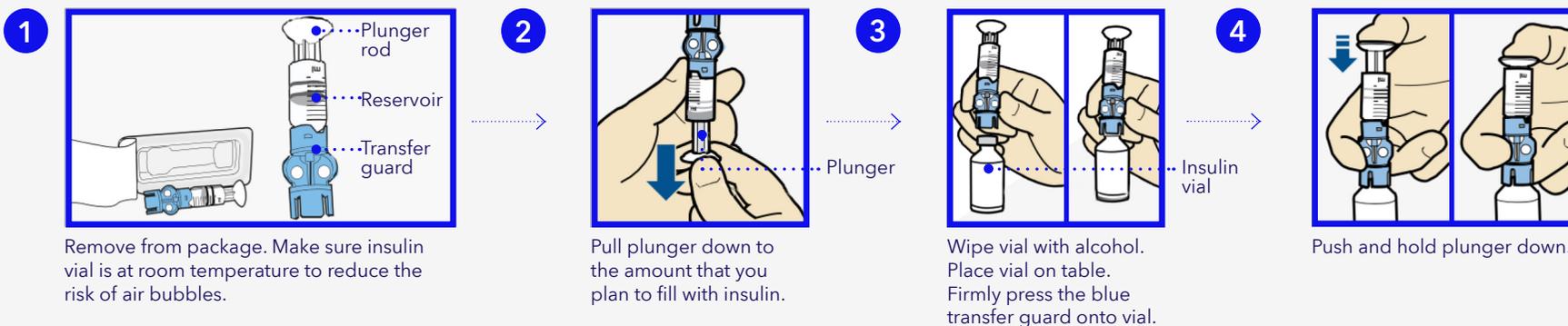


MiniMed™ 780G System changing the MiniMed™ Sure-T® infusion set quick reference guide


! If insulin or any liquid gets inside the tubing connector, it can temporarily block the vents that allow the pump to properly fill the infusion set. This may result in the delivery of too little or too much insulin, which could cause hyperglycaemia or hypoglycaemia.

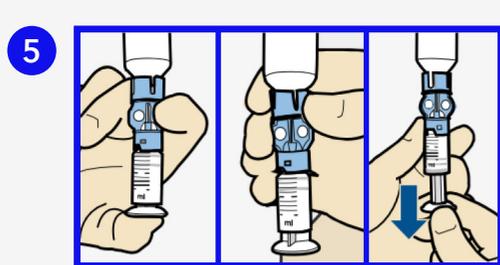

5 With your thumb still on the plunger, flip over so vial is on top. Release thumb and pull plunger down to fill with insulin.

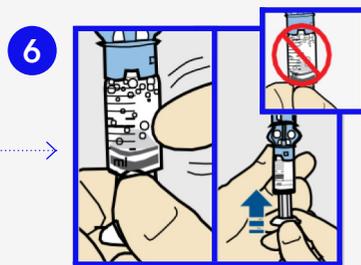

6 Tap the reservoir to move air bubbles to top of reservoir. Push plunger up to move air into vial.

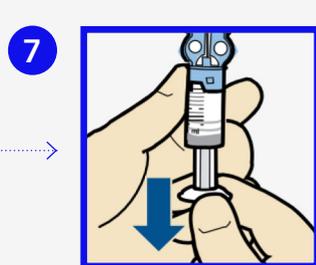


7 If needed, pull plunger back down to amount of insulin needed for 2-3 days.




8 To avoid getting insulin on the top of the reservoir, turn vial over so it is upright. Hold transfer guard and turn reservoir counter-clockwise and remove from transfer guard.

Connect reservoir to infusion set



You will place the tubing connector onto the end of the infusion set to the filled reservoir.

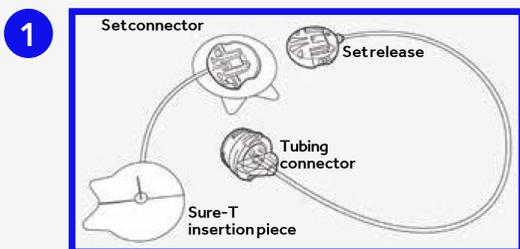


1 Remove the infusion set from packaging.

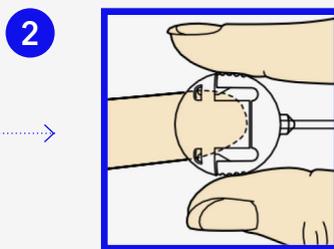


2 Press the edges of the set release connector. With the other hand, pull the cap off.

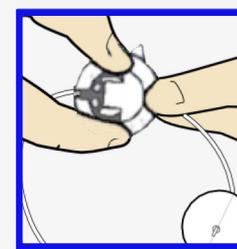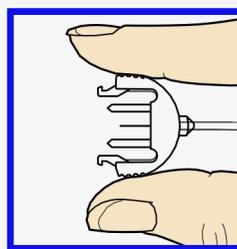


3 Slide the set release connector into the set connector.

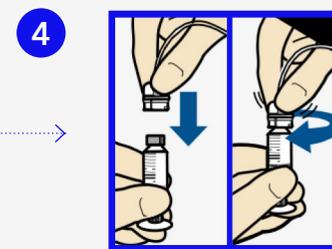

4 Gently push the tubing connector onto reservoir. Turn clockwise until locked. You will hear a click.

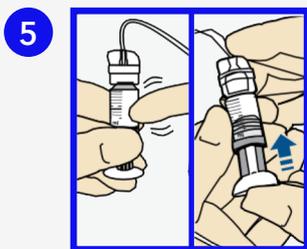

5 If you see air bubbles, tap reservoir to move them to top. Push plunger just a bit to move them into tubing.

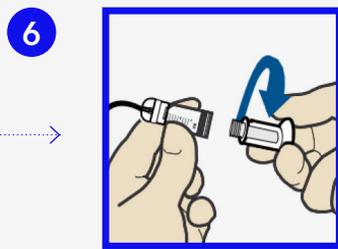

6 Twist the plunger counter-clockwise to loosen and remove.

The backlight may have turned off

Press any button to turn the screen back on.

1 Press **O** to open the menu.

2 Select .

3 Select **Next**.

Place reservoir into pump

Now place the filled reservoir into the reservoir compartment of the pump.

1 Place reservoir into pump.

2 Turn clockwise until you feel reservoir lock into place.

3 Select **Next**.

Load reservoir and fill tubing

Follow these steps to load the reservoir and fill the tubing.

1 Select **Load** and keep holding **O**.

2 When you see this screen, select **Next**.

3 Select **Fill** and keep holding **O** until you see drops at the end of tubing, then let go.

4 After you see drops, press **>** and select **Next**.

Drops at end of tubing

MiniMed™ 780G System changing the MiniMed™ Sure-T® infusion set quick reference guide

Insert infusion set


Next, follow these steps to insert the infusion set into your body.

Fill Cannula?

1. Insert infusion set into body.
2. Select Fill to fill cannula or Done if not needed.


Fill Done

1

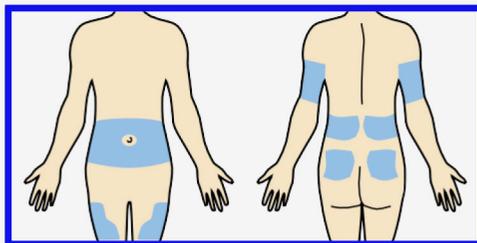


Choose an insertion site from the shaded areas shown here. Wipe with alcohol or other antiseptic.

2

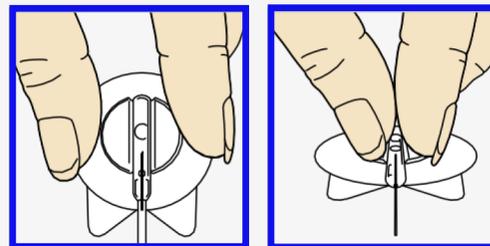


Hold infusion set insertion piece by flipping the plastic wings upwards and pressing them together.

3

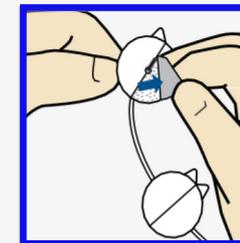


Peel off both pieces of the paper backing from the insertion piece.

4

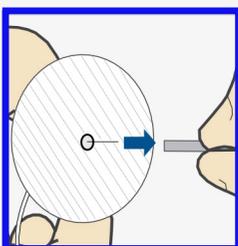


Pull off the needle guard.

5

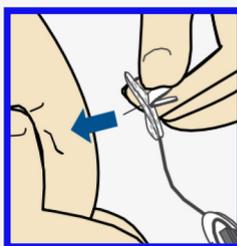


With one hand, pinch up skin at prepared site. With the other hand, insert the needle straight in at a 90 degree angle.

6

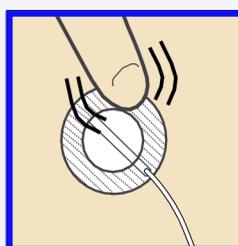

Press adhesive securely to skin.

7

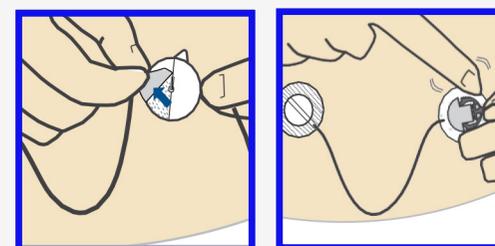

Remove paper backing from the set connector and secure adhesive to skin. Be sure to place it on an area of your body that allows for easy disconnecting from tubing.

Load reservoir and fill tubing

Use the pump to finish the process.

1

Fill Cannula?

1. Insert infusion set into body.
2. Select Fill to fill cannula or Done if not needed.

Fill Done

You do not need to fill a cannula. Press > and select Done.

Your infusion set change is now complete!