


## Setting up & using the CareLink™ Connect app

For Care Partners


- 1 Check the [Compatibility Chart](#) found on your local website FAQ.
- 2 Download the [CareLink™ Connect app](#) on the Apple® or Google Play™ app store.
 

Important - We recommend to turn off your Operating system auto updates to help ensure that you won't be using an unverified version of the app.
- 3 Follow the [On Screen prompts](#).
- 4 Read the [Dosing Disclaimer](#) that states dosing decisions should not be made based on the app then get started.
- 5 Create your [Partner Account](#). Once created proceed to login.
- 6 Send a [Follow Request](#) by inputting the CareLink™ Personal account username of the person you would like to follow.
- 7 [Pump User Approves Request](#) via their CareLink™ Personal account then you are connected.

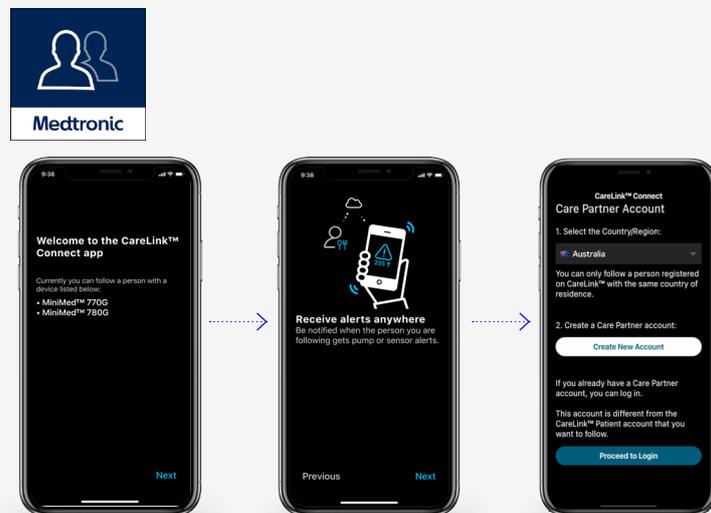


### The app is now ready to use.

You are now connected to your partner.

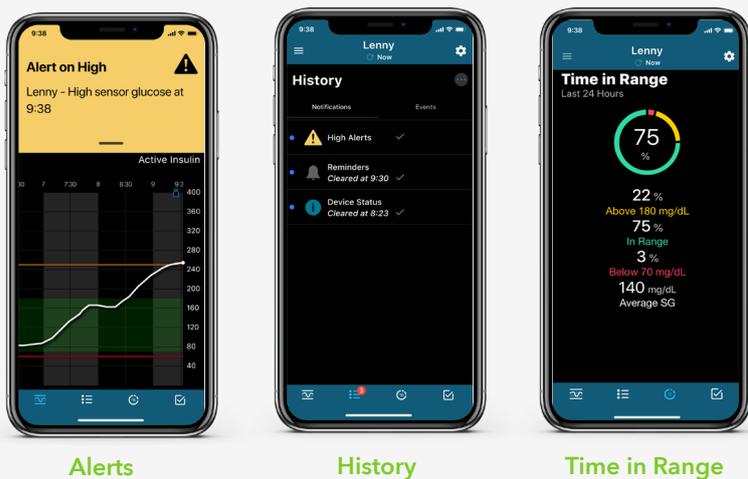

### Home Screen View

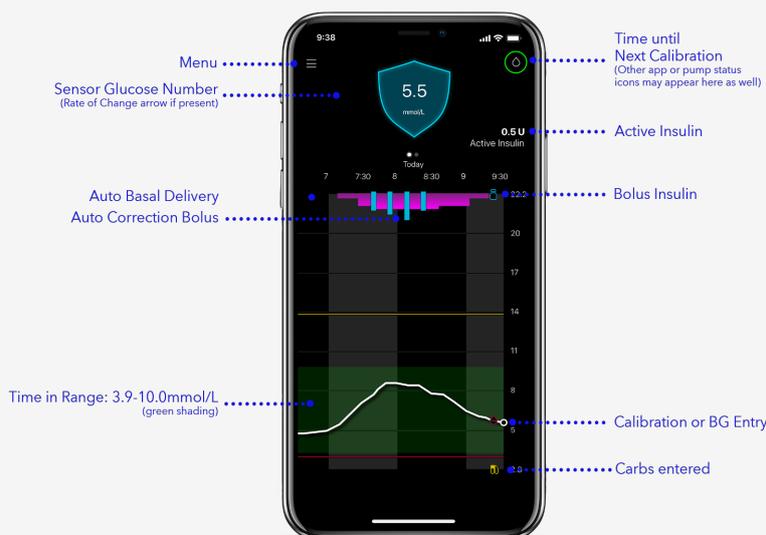

Personalised diabetes insights *anytime, anywhere.*