

## MiniMed™ 780G System at a glance



This quick guide covers the main functions of the MiniMed™ 780G system when the automatic SmartGuard™ function is active. For more details on how to use the product, refer to the system's instruction manual.

### Shortcuts



	To stop the entire insulin delivery or bolus, press <b>UP</b> . You can continue delivery from the same location.	
	You will find the bolus screen by pressing <b>DOWN</b> .	
	You can access the 'Time in Range' by pressing <b>RIGHT</b> .	
	Select Button	
	Notification Light	

You can view the sensor graph by pressing the **DIAMOND BUTTON**.

### Menu

- Press the **SELECT** button on the Home screen
- The Insulin icon will show as selected
- Move to the desired icon: it changes colour and the menu name appears at the top of the screen

<b>HISTORY &amp; GRAPH</b> History Sensor Glucose Review Graph Time in Range	<b>SMARTGUARD</b> SmartGuard Checklist Temp Target SmartGuard Settings SmartGuard On and Off	<b>SOUND &amp; VIBRATION</b> Silence Sensor Alerts Volume Sound Vibration Alert Settings Shortcut
<b>RESERVOIR &amp; SET</b> New Reservoir Fill Cannula	<b>INSULIN</b> Bolus Basal Suspend/Resume Basal Delivery Delivery Settings Shortcut	<b>BLOOD GLUCOSE</b> BG
<b>STATUS</b> Suspend All Delivery SmartGuard Checklist Pump Sensor	<b>PAIRED DEVICES</b> Pair New Device Pair CareLink™ Sensor	<b>SETTINGS</b> Alert Settings Delivery Settings Device Settings

### Sensor calibration

The insulin pump needs to be calibrated at least every 12 hours. Sometimes the pump may also request extra calibration to improve performance. When the pump needs calibration, it displays an **Enter BG now** alert.

- Use only BG values from fingertips for calibration
- See the next page for instructions on measuring your blood sugar

**Note:** If the calibration was not accepted, wait at least 15 minutes. Wash your hands and measure the BG again. If the "Calibration not accepted" alert is still received after the second calibration, a Change Sensor alert occurs.

## Suspending insulin delivery

If the pump is disconnected from the body, insulin delivery needs to be suspended.

**Suspend:** Press UP and select Suspend all Delivery. Select Yes.


**Resume:** Press UP and select Resume Basal. Select Yes.

## Measuring blood glucose

Press the **back button** to send the BG value to the pump without delay.




You can also enter blood glucose to the pump manually:

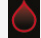1. Choose  from the menu.
2. Enter BG
3. Select SAVE.

## Temporary target

When needed, you can set a temporary target (8.3 mmol/l) for the pump from the **SmartGuard™ menu**. When the temporary target is active, auto correction boluses are not delivered.

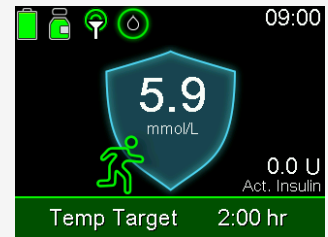


## Pump home screens

**Blue shield:** SmartGuard™ function is active and works normally.




**Red bar:** Insulin delivery is suspended.

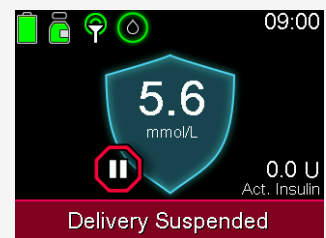


**Note:** If the pump is in manual mode, the shield is not shown on the home screen. The pump also works a little differently. Confirm the instructions for manual mode separately.

## Pump alerts and all alarms

**All alerts, alarms and messages given by the pump must be acknowledged without delay.** If necessary, follow the instructions displayed on the pump screen.

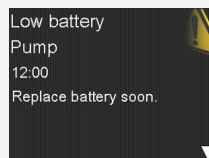

**Alert:** Your pump needs attention.  
*E.g. Low battery. Low reservoir.*

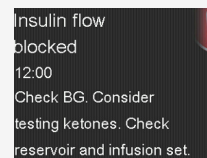

**Alarm:** Address right away.  
*E.g. Insulin flow blocked. Replace battery now.*


**Note:** The notification light flashes when the pump has an alarm or alert.

Staying in the SmartGuard™ feature:

When the pump requires an action to stay in the SmartGuard™ feature, it provides a notification and the time remaining before an exit to the manual mode (max. 4 hours).

- 1) Read the notification from the bar.
- 2) Respond to the action needed.

If the pump exits to manual mode, confirm the instructions separately.

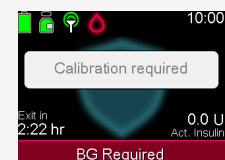

## Medtronic

Medtronic Australasia Pty Ltd  
2 Alma Road  
Macquarie Park, NSW 2113, Australia

Tel: +61 2 9857 9000  
Fax: +61 2 9889 5167  
Toll Free: 1800 777 808

E-Mail: [australia.diabetes@medtronic.com](mailto:australia.diabetes@medtronic.com)  
Facebook: [www.facebook.com/MedtronicDiabetesAUS](https://www.facebook.com/MedtronicDiabetesAUS)  
YouTube: [Medtronic Diabetes ANZ](https://www.youtube.com/MedtronicDiabetesANZ)

<https://www.medtronic-diabetes.com.au/>

**Always read the instructions for use.**

For detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events, please consult the device manual. For further information, contact your local Medtronic representative.  
© 2021 Medtronic Australasia Pty Ltd. All Rights Reserved. Please note: In contacting the Diabetes Toll Free, your personal and health information may be disclosed to an operator located outside Australia. 10250-102021