

Smart MDI system Initiation Settings

Medtronic

Smart MDI including Guardian™ 4 system and InPen™ system

This guide is intended to assist the health care team to provide structured settings advice when setting up the Guardian™ 4 and InPen™ systems.

Guardian™ 4 system settings

Low Alert Settings

Day starts at hours
Low Limit (3.3-5.0 mmol/L) mmol/L
Alert at Low ON / OFF
Alert Before Low ON / OFF
*Time Before Low (10-60 mins) mins

* Only available if Alert Before Low is ON

Night starts at hours
Low Limit (3.3-5.0 mmol/L) mmol/L
Alert at Low ON / OFF
Alert Before Low ON / OFF
*Time Before Low (10-60 mins) mins

High Alert Settings

Day Starts at hours
High Limit (5.5-22.2 mmol/L) mmol/L
Alert at High ON / OFF
Alert Before High ON / OFF
*Time Before High (10-60 mins) mins

* Only available if Alert Before High is ON

Night Starts at hours
High Limit (5.5-22.2 mmol/L) mmol/L
Alert at High ON / OFF
Alert Before High ON / OFF
*Time Before High (10-60 mins) mins

InPen™ system settings

Long-Acting Insulin

Insulin type:

Dose 1 usual amount:

Dose 2 usual amount:

Number of daily doses:

Dose 1 usual time:

Dose 2 usual time:

Rapid-Acting Insulin

Maximum calculated dose: U
Duration of insulin action: h min

Time of day Settings: ON / OFF
Active Insulin Display: ON / OFF

Reminders

Missed dose reminder: ON / OFF
Long-acting insulin reminder: ON / OFF

Check BG: ON / OFF
Replace cartridge: ON / OFF

Carb Counting:

Time of Day				
Target Blood Glucose	mmol/L	mmol/L	mmol/L	mmol/L
Insulin Sensitivity Factor	mmol/L/U	mmol/L/U	mmol/L/U	mmol/L/U
Insulin to Carb Ratio	g/U	g/U	g/U	g/U

Meal Estimation:

Time of Day				
Target Blood Glucose	mmol/L	mmol/L	mmol/L	mmol/L
Insulin Sensitivity Factor	mmol/L/U	mmol/L/U	mmol/L/U	mmol/L/U

	Low Carb	Med. Carb	High Carb
Breakfast	U	U	U
Lunch	U	U	U
Dinner	U	U	U
Snack	U	U	U

Fixed Dose:

mmol/L				
Target Blood Glucose	mmol/L	mmol/L	mmol/L	mmol/L
Insulin Sensitivity Factor	mmol/L/U	mmol/L/U	mmol/L/U	mmol/L/U

Breakfast	U
Lunch	U
Dinner	U
Snack	U

Comments : _____

These instructions are valid until : _____

Prescriber Name : _____ Signature : _____ Date : _____

Call HCP for severe or recurring low BG or if ketones occur. Call Medtronic for technical assistance at 1800 777 808

Medtronic

Medtronic Australasia Pty Ltd
2 Alma Road
Macquarie Park NSW 2113
Tel +61 2 9857 9000
Fax +61 2 9857 5167
Tollfree 1 800 777 808
www.medtronic-diabetes.com.au

Please consult your healthcare professional for a full list of benefits, indications, precautions, clinical results, and other important medical information that pertains to the therapy or products discussed. See the device manuals for detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events. For further information, contact your local Medtronic representative

©2023 Medtronic. All rights reserved. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic.™*
Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.